<b>Control HbA<sub>1c</sub></b> <b>MonlabTest®</b> Turbidimetría latex	 
--	------

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**Quantitative determination of glycated hemoglobin (HbA<sub>1c</sub>)**

Only for professional in vitro diagnostic use  
 Store at 2-8°C

**INTENDED USE**

This product is for the purpose of monitoring accuracy and precision in the quantitative determination of human hemoglobin A1c (HbA<sub>1c</sub>) in blood by automated immunoassay.

**PRODUCT CHARACTERISTICS**

The lyophilized HbA<sub>1c</sub> controls are hemolysates prepared from packed human erythrocytes. The controls provide two levels of HbA<sub>1c</sub>, one level in the normal range and the other level in the elevated range. Stabilizers are added to maintain hemoglobin in the reduced state providing complete control of the HbA<sub>1c</sub> procedure.

**REAGENTS**

<b>HbA<sub>1c</sub> Control Level 1</b>	Human blood. Contains preservatives.
<b>HbA<sub>1c</sub> Control Level 2</b>	The HbA <sub>1c</sub> concentration is indicated on the vial label.

**PREPARATION**

Reconstitute vials with 0.5mL deionized water. Gently mix for 10 minutes. Observe for undissolved material. The reconstituted controls may be dispensed in 0.1mL aliquots, sealed tightly and frozen at -20°C.

The lyophilized HbA<sub>1c</sub> controls should be assayed in the same manner as blood specimens including the hemolysate procedure.

**PRECAUTIONS**

Components from human origin have been tested and found negative for the presence of HBsAg, HCV and antibody to HIV (1/2). However handle cautiously as potentially infectious. Avoid contact with skin and mucous membranes.

**MATERIAL REQUIRED AND NOT PROVIDED**

Hemoglobin A1c Reagent kit, from Monlabtest Ref: MO-165035  
 Hemoglobin A1c Calibrator kit, from Monlabtest Ref: MO-165061

**STORAGE AND STABILITY**

1. Store controls at 2-8°C. Stable until expiration date if sealed tightly. PROTECT FROM LIGHT AND HEAT.
2. Reconstituted controls retain their assigned values for at least three months if frozen. If not frozen, the reconstituted controls are stable at least one month if stored at 2-8°C and sealed tightly.
3. Do not freeze and thaw more than once.

**PROCEDURE**

To be used in turbidimetric assays.

**PRESENTATION**

MO-165062	Level 1	1 x 0,5 mL
	Level 2	1 x 0,5 mL